







### INSTRUCTIONS

- |   |   |
|---|---|
|    | <p>1. Wait 15 minutes after 1st alcoholic drink ( as alcohol takes upto 15 minutes to enter your system) OR have a glass of water before taking the test.</p>   |
|   | <p>2. Squeeze the middle of the outer plastic tube between thumb and forefinger to break the inner glass ampule containing the yellow crystals.<br/><b>SQUEEZE ONLY ONCE.</b><br/><b>DO NOT CRUSH OR BEND TUBE.</b><br/>Test user immediately.</p>  |
|  | <p>3. Take a deep breath and <b>BLOW VERY HARD</b>, continuously through designated end of tube, (indicated by arrow) for 12 seconds (preferably in one breath).<br/><b>EXHALE INTO TUBE DO NOT INHALE.</b></p>   |
|  | <p>4. Shake test to distribute crystals evenly in viewing window. Lay test on a flat surface for 2 minutes. Yellow crystals will change colour if alcohol is present. When crystals turn to a white or grey shade alcohol is at or above the level being tested for. To confirm you can compare crystal colours to those of an unused test.</p> |

Note: Accuracy of test results may not be reliable if test is not conducted according to instruction.  
Please refrain from driving after drinking.  
Do not drive at all if any crystals change colour.

SureScreen Diagnostics Ltd.  
1 Prime Parkway, Derby DE1 3QB  
Tel: 01332 365318 Fax: 01332 292230  
Email: sales@surescreen.co.uk