





SureScreen Diagnostics Ltd

1 Prime Parkway
Derby, DE1 3QB
T: +44 (0)1332 365318
F: +44 (0)1332 292230
E: info@surescreen.com
www.surescreen.com


Strip Tests

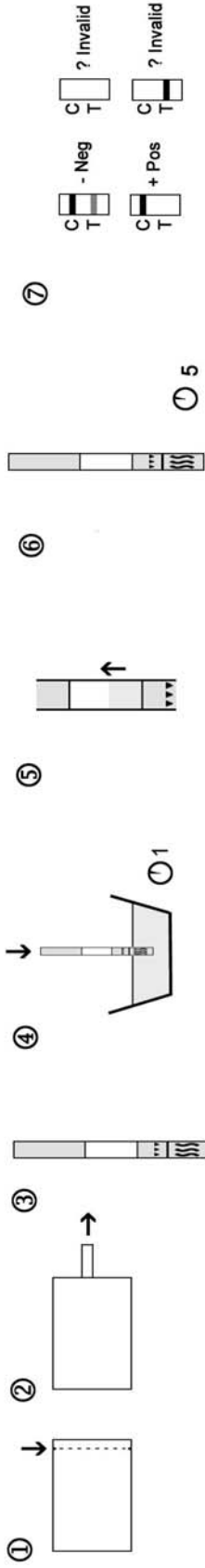


Cassette Tests

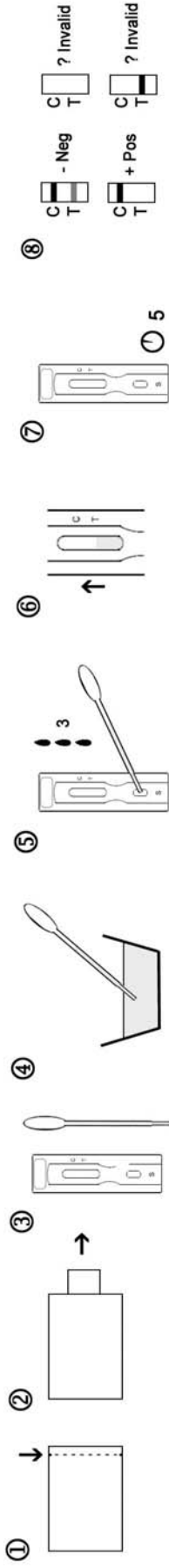

Multicassette Tests

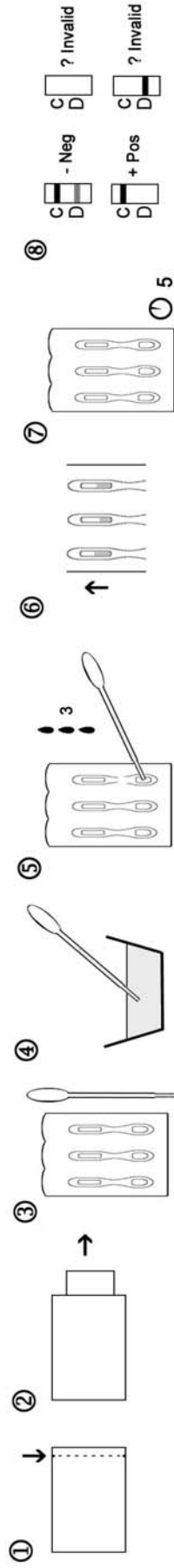

Multipanel Tests

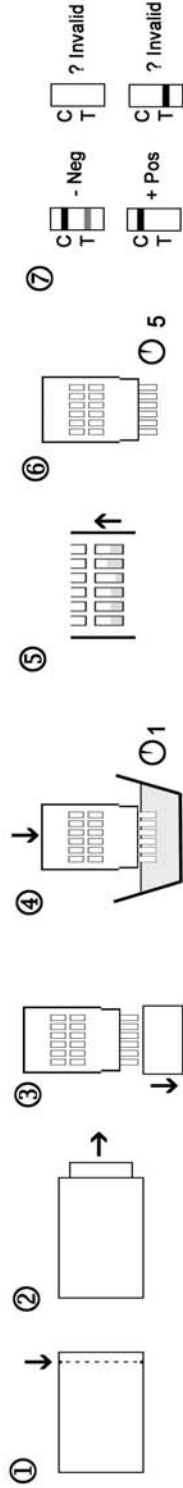

Dip & Read



Drip & Read





Drug Type	Code	Cut-off
Amphetamines	AMP	1000ng/ml
Barbiturates	BAR	300ng/ml
Benzodiazepines	BZO	300ng/ml
Cannabis	THC	50ng/ml
Cocaine	COC	300ng/ml
Dextropropoxyphene	PPX	300ng/ml
MDMA (ecstasy)	MDMA	500ng/ml
Methadone	MTD	300ng/ml
Methamphetamines	mAMP / MET	1000ng/ml
Opiates (high sensitivity)	MOP	300ng/ml
Opiates (low sensitivity)	OPI	2000ng/ml
Phencyclidine	PCP	25ng/ml
Tricyclics	TCA	1000ng/ml



In vitro diagnostic use only



Single use only



Please read instructions before use



Precautions

- For healthcare professionals including professionals at point of care sites.
- For professional in vitro diagnostic use only. Do not use after expiration date.
- The test device should remain in the pouch until use
- All test specimens should be considered potentially hazardous and handled in the same manner as an infectious agent.

Limitations

- SureScreen rapid tests provide a qualitative preliminary result only. A secondary analytical method must be used to obtain a confirmed result.
- It is possible that technical or procedural errors, as well as interfering substances in the urine specimen may cause erroneous results.
- SureScreen drug tests do not differentiate between drugs of abuse, medications or certain foods or food supplements.

Cross reactivity data can be found at www.surescreen.com